

ATTENDS® PERSONAL CARE’S RECOMMENDED PRODUCTS

PRODUCT CATEGORY	PRODUCT NAME	SIZE	CATALOG NUMBER
	Attends® Discreet Women's Ultra Thin Pads	NA	ADPTHIN
	Attends® Discreet Women's Moderate Pads		ADPMOD
	Attends® Discreet Women's Maximum Pads		APDMAX
	Attends® Discreet Women's Maximum Long Pads		ADPMAL
	Attends® Discreet Women's Ultimate Pads		ADPULT
	Attends® Overnight Underwear	M, L, XL	APPNT20, APPNT30 APPNT40
	Attends® Bariatric Underwear	XXL, XXXL	AU50, AU60
	Attends® Underwear, Ultimate Absorbency	Y/S, M, L, XL	APP0710, APP0720, APP0730, APP0740
	Attends® Underwear, Maximum absorbency	M, L, XL	AP0720, AP0730, AP0740
	Attends® Underwear, Extra Heavy Absorbency	M, L, XL	APV20, APV30, APV40
	Attends® Overnight Briefs, Overnight Absorbency	M, L, XL	DDEW20, DDEW30, DDEW40*
	Attends® Briefs, Ultimate Absorbency	M, L, XL	DDC20, DDC25, DDC30, DDC40*
	Attends® Stretch Briefs, Ultimate Absorbency	S/M, L/XL	DDSMR, DDSLXL
	Attends® Bariatric Briefs, Ultimate Absorbency	2XL, 3XL 4XL, 5XL	DD50, DD55, DD60, DD70
	Attends® Briefs, Youth/Small	Y/S	DD10
	Attends® Briefs, Maximum Absorbency	M, L, XL	DDA20, DDA30, DDA40
	Attends® Brief, Extra Heavy Absorbency	M, L, XL	DDP20, DDP30, DDP40

Selecting the Best Absorbent Products for Your Patients

Body-worn products are the most prevalent strategy for continence management in adults with urinary or fecal incontinence. To find the best information in the industry on these products, the WOCN held a peer reviewto identify the important considerations for body worn products.

The outcome? The Wound Ostomy & Continence Nurses Society Journal’s (JWOCN) peer-reviewed article, **Assessment, Selection, Use and Evaluation of Body-Worn Absorbent Products for Adults with Incontinence**, that provides **pure, clinical evidence to help guide facilities and caregivers.**

It’s important to remember ... There is no single answer for everyone. Follow this summary to help you choose the right products for a pathway to the **best outcomes for your patients!**



PATIENT ASSESSMENT




Potential outcomes when following these guidelines include:

- A good night's sleep with less interruptions
- Lower skin pH levels due to better fit and improved breathability
- Lower chances of incontinence-associated dermatitis due to a healthy microclimate and less humidity
- Lower likelihood of pressure ulcers
- Enhanced personal dignity
- Increased ability to interact with others while containing and concealing incontinence
- Higher health-related quality of life

BODY WORN PRODUCT RECOMMENDATIONS



PANTY LINERS & PADS

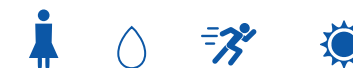

"In women with light incontinence, inserted, disposable pads designed for containment of urine are less likely to leak and are ranked higher by users ..."




UNISEX PULL-UPS



"In general, [underwear] should be selected in preference to inserts to minimize leakage. Unisex pull-ups are a baseline recommendation for moderate - heavy users with dual daytime incontinence."

Men and women leak at different volume-outputs. Consider absorbency level of the product to ensure a best fit.

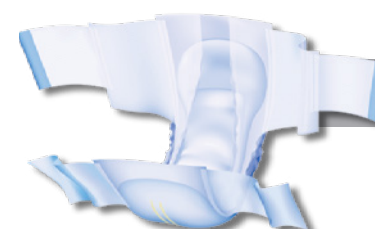

DISPOSABLE BRIEFS



"... strongly recommended selecting a brief with a seal/reseal mechanism that provides not only containment but also safety for the patient and caregiver."

"Contemporary designs provide variable degrees of breathability, allowing airflow to the inner aspect of the product and the user's skin without compromising containment of liquid within the product."

1. "Assessment, Selection, Use, and Evaluation of Body-Worn Absorbent Products for Adults with Incontinence", J Wound Ostomy Continence Nurs. May/June 2018
 2. "The Impact of Microclimate on Skin Health With Absorbent Incontinence Product Use: An Integrative Review", J Wound Ostomy Continence Nurs. July/August 2018